culla-chair by elmi

< therapy and relax

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We would like to thank you for your trust and for the purchase of our product. **culla-balance** is outstanding for its quality, its safety, its simple use and its modern design. In order to use this item properly, please read these user's instructions very carefully.

**Dear Customer,
Please read this user's manual carefully before using culla-chair for the first time!**

INDEX

Safety instructions	30
Warrenty and product liability	31
Intended use	31
Components Culla Care	32
Components Culla Balance	33
Components Culla Dorma and Culla outdoor	34
Adjustment and operation	35
Technical data Culla Care	36
Technical data Culla Balance	37
Technical data Culla dorma	38
Types	39
Cleaning instruction and accessories	40-41
Assembly	41
Maintenance and technical data	42
Spare parts	42

SAFETY INSTRUCTIONS

Culla chair is provided with the CE-marking and conforms to the requirements as laid down by the Council Directive 93/42/EEC

Norm UNI EN 12182 · Norm UNI En 1985 · Norm UNI EN ISO 14971

It has been developed following the latest regulations. The manufacturing process emphasizes continuous quality control.

Weight limit: 150 kg max user weight

Precautions!

Culla chair should only be used if a qualified person or a family member is present.

Culla chair should only be used by people who regarding their height and body constitution are able to assume a lying and seated position.

Culla chair are suitable for indoor and outdoor use.

Please remove any objects from furniture that can be knocked over or fall down while using **culla-chair**. The version **culla chair indoor** is suitable for indoor use and for outdoor use only on completely even ground. The version **culla chair outdoor** is suitable for indoor use and also for outdoor use due to its big rear wheels and its safety brake.

Culla chair can only be used by people who are familiar with the instructions and applications of this user manual.

Please check that in the user's area of **culla chair** there are no obstacles such as stairs, steps or items of any kind lying on the floor that could cause **culla chair** to overturn.

Do not carry out any movements or interference, if you are not absolutely sure of their effects and consequences; in case of the slightest doubt please contact the nearest service centre or the manufacturer directly.

The application of the use of **culla chair** must be carried out with the support of qualified personnel. Pay attention to possible points of danger!

The use of **culla chair** and its accessories should be done under medical supervision. Do not use **culla chair** on uneven ground, on slopes or on stairs.

The use of **culla chair** by unauthorized people – even if it is only for playing – is forbidden. When stopping always block the brakes, even on level surfaces.

Before every use, check that the brakes function correctly, check that the casters have not become worn, and that all the accessories and components that could get loose are adjusted and properly tightened.

Do not systemize or leave **culla chair** on steep or uneven ground.

Do not use **culla chair** if parts are faulty or missing. Before use check that, all the fasteners of **culla chair** are tightened.

Culla chair should be cleaned as described in the „cleaning“ chapter.

In case of an accident or defects, contact your dealer who delivered **culla chair**. Please keep these user's instructions for further consultation.

Culla chair are has to undergo service annually to assure long-term use by avoiding damage to the product and the individual. An authorized person should carry out the servicing.

WARRANTY AND PRODUCT LIABILITY

Elmi GmbH/srl grants a two years' warranty on culla chair.

Only original spare parts and accessories must be used. The guarantee becomes void if repairs or changes to culla chair or on its components are undertaken by unauthorized personnel.

Every product responsibility of the manufacturer (Elmi Srl, Zona Artigianale 25, I-39052 Caldaro) is in this case excluded beforehand.

The manufacturer is not responsible for damages to persons, animals or things in the following cases: improper use of culla chair, use of culla chair by unauthorized persons, inappropriate applications of parts or accessories, changes or interventions not previously authorized by the supplier, usage of no-original spare parts, no-observance of the instructions in this user's manual, exceptional incidents.

INTENDED USE

...the second cradle

...more comfortable than a cradle...

What can the patient expect?

- Cosiness** culla chair is a therapy- and relax chair. It offers the patient cosiness, relax and warmth.
- Mobility** culla chair is easily manoeuvrable and with its blockable caster wheels extremely mobile. The sitting/lying position is easily regulated.
- Security** eh embryo posture gets relived. Its nest-like shape adapts to the body, it gives sustain, safety and nest-like warmth.
- Comfort** The perfect form and the high quality upholstery secure an optimal position from sitting to lying. The comfortable front access facilitates the active transfer.

Who needs culla chair?

Culla chair is a relief for people with physical and mental disabilities as well as for elderly users. It helps people with muscular problems, with decubitus and it calms down patients suffering from Alzheimer and Parkinson.

Culla chair improves the quality of life at home, in nursing homes, retiring homes and hospitals.

Culla chair was designed and made to offer the patient cosiness and relax; the patient is back in his childhood and the cosy cradle shape of culla chair gives the user a sensation of being protected. He definitely calms down.

Culla chair can only be used for the applications mentioned above. The manufacturer assumes no responsibility for other applications and for damages caused by improper use.

We wish you a good success with the use of **culla chair!**

COMPONENTS CULLA-CHAIR

COMPONENTS CULLA-BALANCE

BALANCE

without removable armrest

BALANCE UNO

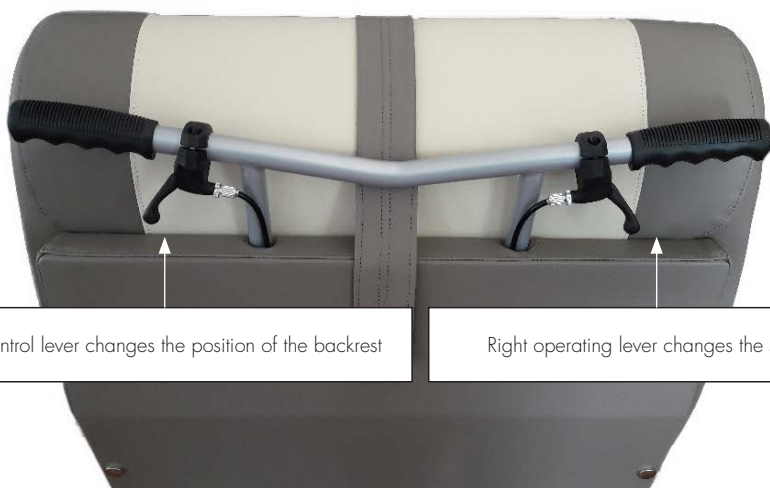
with one removable armrest

BALANCE DUO

with two removable armrests


ADJUSTMENT CULLA-DORMA

Left control lever changes the position of the backrest

Right operating lever changes the seat tilt

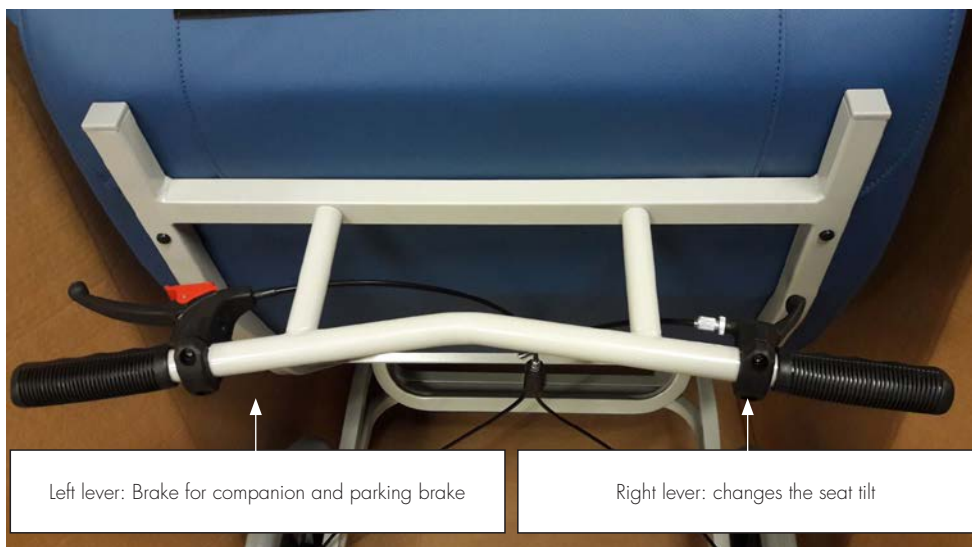ADJUSTMENT CULLA-CHAIR OUTDOOR

Left lever: Brake for companion and parking brake

Right lever: changes the seat tilt

ADJUSTMENT AND OPERATION

fig. 1

fig. 2

fig. 3

fig. 4

fig. 5

fig. 6

fig. 7

fig. 8

ress the inclination lever and put **culla chair** in the upright position for a perfect access. (fig. 2). Approach the patient to guarantee an easy access (fig. 1).

Please pay attention that the footrest is under the seat (fig. 3). To position the footrest use the side lever. Loosen the handle, after reaching the desired height tighten the handle again (fig. 4).

When the patient is sitting comfortably (fig. 5), you can press the inclination lever and put the patient in a position ranging from sitting to lying (fig. 2).

Find the best position in height and inclination. Positions are adaptable to the needs of every single user. The adjustment of the back and the seat remain in an orthopaedic and anatomic correct position independently from inclination of the back (fig. 6).

Put the footrest in the desired height and inclination by loosening the side handle (fig. 4). Tighten it again after reaching the desired position.

The lateral transfer is also possible through the removable armrests.

TECHNICAL DATA CULLA-CHAIR

indoor

outdoor

TECHNICAL DATA CULLA-BALANCE

TECHNICAL DATA CULLA-DORMA

BASIS MODELS CULLA-CARE

culla-care CC4007 therapy chair and day bed **culla-chair indoor**

- colour upholstery green

culla-care CC4006 therapy chair and day bed **culla-chair indoor**

- colour upholstery red

culla-care CC4008 therapy chair and day bed **culla-chair indoor**

- colour upholstery yellow

culla-care CC4003 therapy chair and day bed **culla-chair indoor**

- colour upholstery blue

culla-care CC04011 OUTDOOR therapy chair and day bed with big rear wheels and brake

- colour upholstery yellow

culla-care CC04010 OUTDOOR

- colour upholstery blue

culla-care CC04009 OUTDOOR

- colour upholstery green

culla-care CC04008 OUTDOOR

- colour upholstery red

BASIS MODELS CULLA-BALANCE

culla-balance (without removable armrest)

balance CB4009 therapy chair and day bed **culla-balance**

indoor colour upholstery: taupe/mandarin

balance CBO4012 therapy chair and day bed **culla-balance**

outdoor colour upholstery: taupe/mandarin

culla-balance transfer (with one removable armrest)

balance transfer CBT4010 therapy chair and day bed **culla-balance transfer**

indoor colour upholstery: taupe/mandarin

balance transfer CBTO4013 therapy chair and day bed **culla-balance transfer**

outdoor colour upholstery: taupe/mandarin

culla-balance transfer duo (with two removable armrests)

balance transfer duo CBT4011 duo therapy chair and day bed **culla-balance transfer duo**

indoor colour upholstery: taupe/mandarin

balance transfer duo CBTO4014 duo therapy chair and day bed **culla-balance transfer duo**

outdoor colour upholstery: taupe/mandarin

culla dorma CDT4016 therapy chair and day **culla-dorma**

indoor colour upholstery: taupe/mandarin

CLEANING INSTRUCTIONS ACCESSORIES

CC5000	artificial lamb fur cover washable (follow washing instructions)
CC5001	terry towel cover washable (follow washing instructions)
CC5003	side-table inclinable with fixture
CC5004	infusion stand with fixture
CC5005	reduction cushion as side support 1 piece
CC5006	reduction cushion to reduce height and depth

Please find more information in our catalogue!

It is strictly forbidden to make any tilt adjustments or change parts or accessories with the user in the therapy chair **culla-chair**!

Cleaning instructions – seat: clean with damp soapy cloth and rinse well with clean water. Do not use solvents, bleaches or synthetic detergents .

Cleaning instructions – metal parts: keep all metal and upholstered parts clean and wipe them down with a soft and damp cloth and dry.

Cleaning instructions – caster wheels: keep the caster wheels clean; remove dust, hairs and other substances careful .

ACCESSORIES

Together with culla-chair the following accessories are available:

- CC5003 side-table inclinable with fixture (fig. A)
- CC5004 infusion stands with fixture (fig. B)
- CC5005 side support 1 piece (fig .D)
- CC5006 cushion to reduce height and depth (fig. C)
- CC5005 side support 1 piece (fig .D)
- CC5006 cushion to reduce height and depth (fig. C)

fig. A: Mount on the frame and screw up tightly.

fig. B: Mount on the frame and screw up tightly.

fig. C Place the conical reduction cushion in the therapy chair to reduce height and depth.

fig. D: Place the side support left or/and right in the therapy chair to support the pelvis and to reduce superfluous space at the sides.

After all these preparations it is time to start with the introduction to the use of **culla-chair**.
No pre-regulations made!

It is important to get the patient to know the aid and its functions. Normally patients are not afraid of using it; very often they react with enthusiasm and pleasure. Persons who tend to have vehement and uncontrollable movements should never be left unattended for safety reasons.

For „anxious“ people, who are afraid of using culla-balance, the following procedure is advisable: when the patient is trying **culla-chair** for the first time he should not be left alone, but should be accompanied by qualified personnel or the therapist and a family member. They place themselves in front, next to or behind the patient, they convince the patient to sit down carefully and then move **culla-balance** slowly and with caution.

These cautionary measures give the patient a great sense of security, especially if at the same time he gets an explanation of what is happening. Initially it is necessary to try to reach a movement by pushing at constant speed. Limited distances, whether for “daring” or for “anxious” patients, are advisable, as they produce a useful effect of orientation and progress

LATERAL TRANSFER

culla-chair has a removable height-adjustable headrest that allows a subjective setting of the device and functions as a backrest extension. The shoulder section is larger to give more freedom. The footrest has an ergonomic design which supports the calves. The comfortable front-side entrance facilitates the transfer. The pleasant concave shape adapts to the body offering support, security, warmth, and replicates the embryonic position.

The removable armrest allows easy and safe lateral patient transfer without help of lifters, thus avoiding physical discomforts for the attendant.

ASSEMBLY

culla-chair functions in a very easy and secure way: even though a good introduction by the dealer/therapist is necessary. There are instructions to be considered for the optimal use of this device.
Start-up:

culla-chair is delivered in one part. The single parts are: base with caster wheels and brakes, upper part with nest-like seating, head cushion, reduction cushions if necessary, footrest. The soft cover in terry towel, lamb's wool or artificial lamb fur can be put over the seating and easily fixed with velcro at the back on two sides and in front.

MAINTENANCE AND TECHNICAL DATA

Wheel dimensions:

wheel dimensions front: 125 x 30 mm indoor, 125 x 30 mm outdoor

wheel dimensions rear: 150 x 30 mm indoor, 400 x 40 mm outdoor

Max. user weight: 150 kg

Accessories

For any substitutions of accessories please refer to the instructions. Remember that for dismounting follow the instructions in inverse order.

Maintenance:

Maintenance has to be performed by the dealer or its representative once a year.

SPARE PARTS

Never make any substitutions of parts or accessories while the patient is using **culla-chair**!

Any product liability from the side of the manufacturer (Elmi Srl, Zona Artigianale 25, 39052 Caldaro, BZ, Italy) is excluded from the beginning in such cases.

For questions on spare parts and accessories please contact your authorized dealer or the manufacturer directly.

tel: 00390471961002 or via e-mail: info@elmi-reha.it.

Environmental factors:

All packing materials are recyclable, as carton and paper are used.

If plastic foils are eventually used they are made in PE and therefore also recyclable.

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(If possible, use the maintenance code at the bottom of the page when filling out)

	Name / nome / name:	Datum / data / date:
Produkt Prodotto product Serien Nr. / n. seriale / serial n.		
Kabel Neigungs cavo basculamento cable tilt adjustment		
Gasdruckfeder Sitz molla a gas seduta pneumatic spring seat		
Gelenke Fußablage snodi poggiapiedi link footrest		
Bedienhebel Neigung leva basculamento control lever tilt		
Bremshebel leva freno brake lever (outdoor)		
Schiebegriff maniglione di spinta push handle		
Sitz / seduta / seat		
Bespannung / rivestimento / cover		
Vorderräder / ruote anteriori / front wheels		
Hinterräder / ruote posteriori / rear wheels		
Rahmen / telaio / frame		